



EduLink: Replacing the Earpiece Cover (Wax Filter)

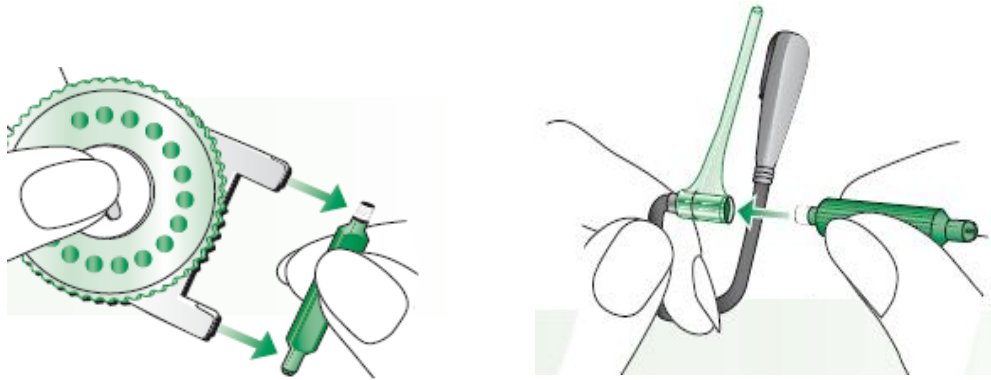

Detach replacement tool from the dispenser. Push the **THREADED** end of the tool into the earpiece cover. Twist the thread to about 1mm, until resistance is felt, then carefully withdraw the tool with the earpiece cover attached.



Press the replacement tool with the soiled earpiece cover into the center of the dispenser. Slide the tool sideways into the slot and pull away from the dispenser. Turn the dispenser wheel to open one of the chambers. Press the **TAPERED** end of the tool (not the threaded end) into the fresh earpiece and withdraw it from the dispenser.

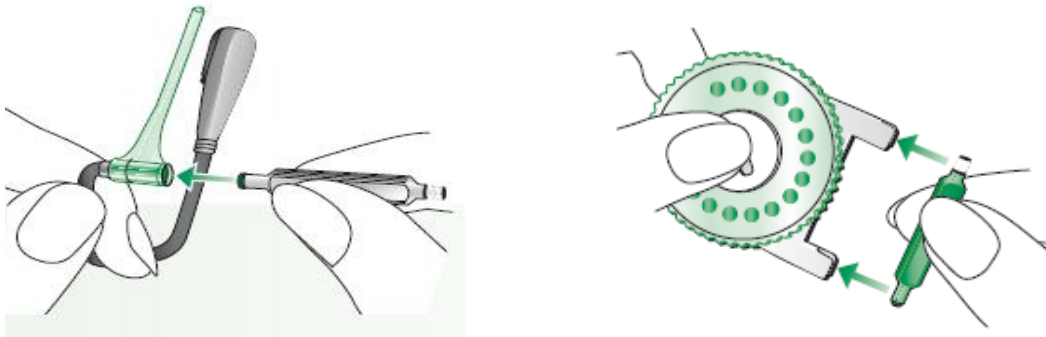

Press the earpiece cover into the sleeve. Withdraw the replacement tool and refit into the dispenser for safekeeping.