

# iSense Micro Receiver



- ① iSense Micro housing
- ② ON/OFF switch and battery compartment (312 battery)
- ③ Retention piece
- ④ Flex
- ⑤ Ear wax filter


## Switching ON

Close the battery compartment by pressing on the base of the instrument.



## Switching OFF

Using the nail grip, slightly open the battery cover until it snaps into its OFF position.

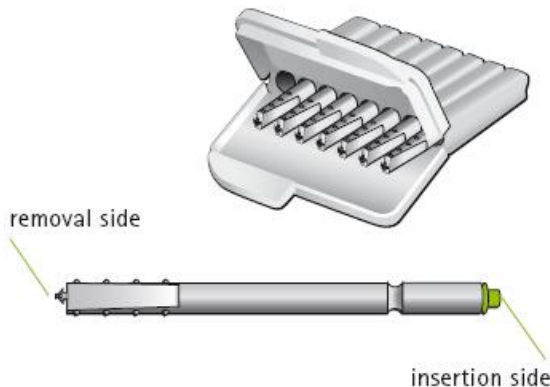

- If the wax filter is blocked and needs to be replaced, the following sounds will be heard:  
Beep – Beep – Beep...  
Beep – Beep – Beep...  
Three beeps every five seconds.